



Proximal Lateral Humeral Right Jig 00-2360-092-01

Assembly/Disassembly
Instructions



Disassembly Instructions for Proximal Lateral Humeral Right Jig

1. Unthread the Jig Set Screw **2** out of the Proximal Lateral Humeral Jig **1**.
2. End of disassembly.

Assembly Instructions for Proximal Lateral Humeral Right Jig

1. Orient the flat surface of the Proximal Lateral Humeral Jig **1** facing the Jig Set Screw **2** threads.
2. Thread the Jig Set Screw **2** into the Proximal Lateral Humeral Jig **1** fully until it moves freely inside the jig.
2. End of assembly.



+H124970000513001/\$100610F10M

97-0000-513-00 1003-T20 Printed in USA ©2010 Zimmer, Inc.