




## Distal Lateral Tibial Left Jig

00-2360-095-02

Assembly/Disassembly  
Instructions

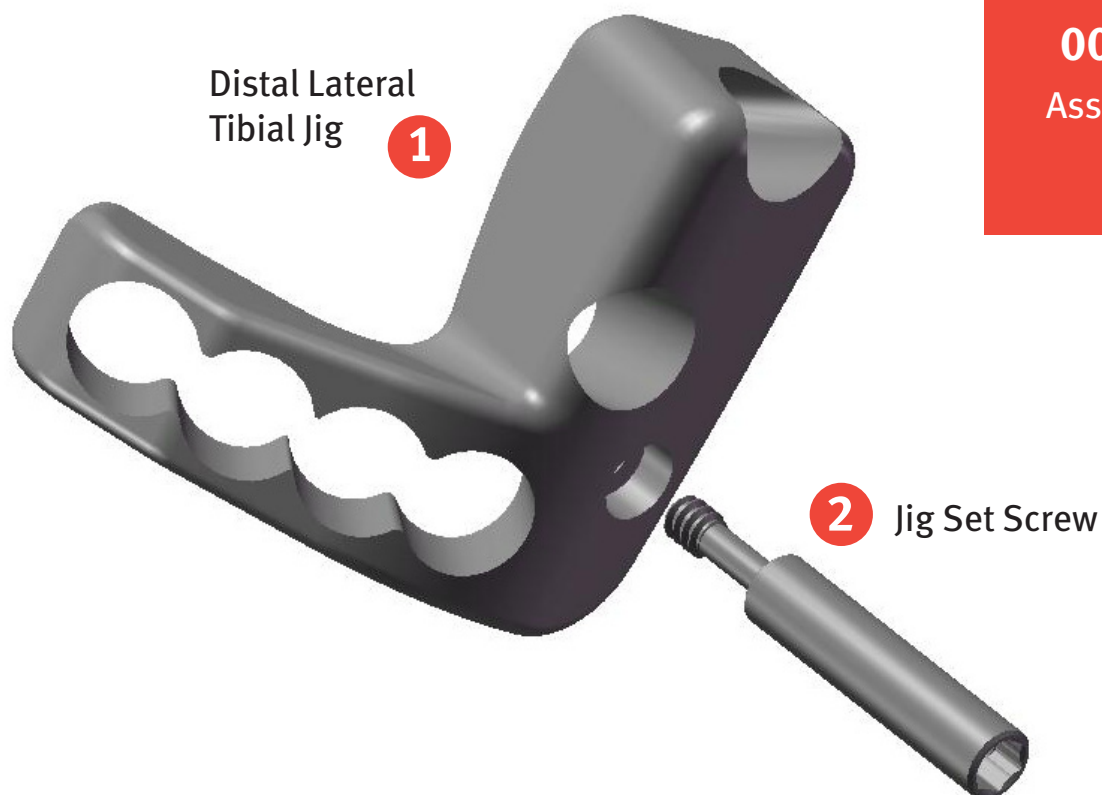

Distal Lateral  
Tibial Jig **1**

**2** Jig Set Screw

## Disassembly Instructions for Distal Lateral Tibial Left Jig

1. Unthread the Jig Set Screw **2** out of the Distal Lateral Tibial Jig **1**.
2. End of disassembly.

## Assembly Instructions for Distal Lateral Tibial Left Jig

1. Orient the flat surface of the Distal Lateral Tibial Jig **1** facing the Jig Set Screw **2** threads.
2. Thread the Jig Set Screw **2** into the Distal Lateral Tibial Jig **1** fully until it moves freely inside the jig.
2. End of assembly.



+H124970000518001/\$100610F10R

97-0000-518-00 1003-T40 Printed in USA ©2010 Zimmer, Inc.