




Modified MIS Femoral Inserter/Extractor with Strike Plate

Assembly/Disassembly Instructions

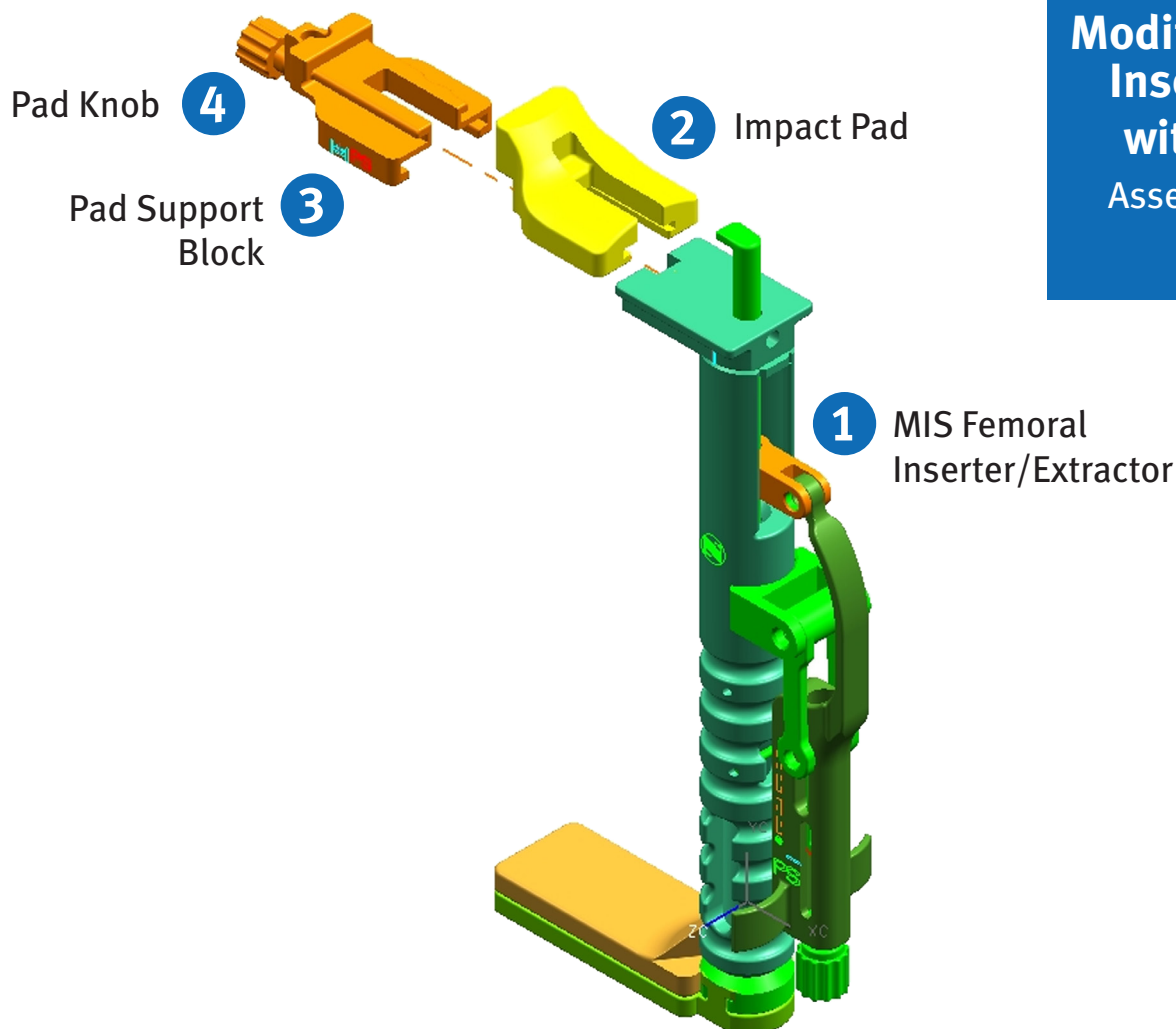

Disassembly Instructions for 82-0153-008-00

1. Unscrew the Pad Knob **4** then remove Pad Support Block **3** from the MIS Femoral Inserter/Extractor **1**.
2. Slide Impact Pad **2** from Pad Support Block **3**.
3. End of disassembly.

Assembly Instructions for 82-0153-008-00

1. Slide Impact Pad **2** onto Pad Support Block **3**.
2. Screw the Pad Knob **4** onto the MIS Femoral Inserter/Extractor **1**.
3. End of assembly.



+H124970000402001/\$100610F10J

97-0000-402-00 1003-K18 Printed in USA ©2010 Zimmer, Inc.

